

BrightDAB, substrate DAB
(For use with HRP-labeled detection systems)

Instruction For Use

These instructions apply to the WellMed BrightDAB substrate.

1. Intended Use
2. Summary and explanation
3. Kit components
4. Availability
5. Usage
6. Recommended Staining Protocol
7. Control slides
8. Storage
9. Warnings and precautions
10. Troubleshooting
11. Reference

1: Intended Use

For in-vitro Diagnostic Use.

WellMed BrightDAB substrate DAB is intended for use in immunohistochemistry.

2: Summary and explanation

WellMed BrightDAB substrate DAB (3,3'-Diaminobenzidine) is a widely used chromogen for immunohistochemical staining and immunoblotting. When in the presence of peroxidase enzyme, DAB produces a brown precipitate that is insoluble in alcohol and xylene. This product comes in a two-component system consisting of a liquid stable DAB chromogen and DAB substrate buffer. DAB Solution A: Ready-to-Use Buffered H₂O₂; DAB Solution B: Concentrated DAB solution. BrightDAB is a very stable and superior formulation of DAB. In some cases, antibodies titers may increase by two-fold. BrightDAB can be used both manually and on automated stainers.

The clinical interpretation of any staining or its absence should be determined by a qualified pathologist and complemented by morphologic studies; controls should be evaluated within the context of the patient's clinical history and/or other diagnostic tests.

3: Kit components

BrightDAB, substrate DAB; consists of two components:

1. DAB Solution A: Buffered H₂O₂ (Ready-to-Use)
2. DAB Solution B: Concentrated DAB solution

4: Availability

Catalog Number	Contents	Volume
BS04-110	BrightDAB, substrate DAB <ul style="list-style-type: none"> • Sub cat.no.: BS04-110A: DAB Solution A: Buffered H₂O₂ (Ready-to-Use) • Sub cat.no.: BS04-110B: DAB Solution B: Concentrated DAB solution 	110 ml 5 ml

BS04-500	BrightDAB, substrate DAB	
	<ul style="list-style-type: none"> Sub cat.no.: BS04-500A: DAB Solution A: Buffered H₂O₂ (Ready-to-Use) Sub cat.no.: BS04-500: DAB Solution B: Concentrated DAB solution 	500 ml 22 ml
BS04-999	BrightDAB, substrate DAB	
	<ul style="list-style-type: none"> Sub cat.no.: BS04-999A: DAB Solution A: Buffered H₂O₂ (Ready-to-Use) Sub cat.no.: BS04-999B: DAB Solution B: Concentrated DAB solution 	1000 ml 45 ml

5: Usage

- Working solution: Add 40 µl DAB Solution B (± one drop) to 1 ml Solution A, mix well.
- Incubate the DAB solution one time 8 minutes without washing in between.

6: Recommended Staining Protocol

Step	Reagent	Template step	Incubation time
1	Deparaffinize and rehydrate tissue section	Slide/tissue preparing	-
2	Wash aqua dest	Wash	2x 5 min
3	If applicable; HIER or digestive enzyme	Pre-treatment	*
4	Wash buffer	PBS or TBS buffer	2x 5 min
5	H ₂ O ₂ (concentration 3%)	Tissue preparing	10 min
6	Wash buffer	PBS or TBS buffer	2x 5 min
7	Primary antibody	Antibody	30 min
8	Wash buffer	PBS or TBS buffer	2x 5 min
9	Detection system, polymer HRP	Labeled polymer	30 min
10	Wash buffer	PBS or TBS buffer	2x 5 min
11	Substrate	DAB	8 min
12	Wash aqua dest	Wash	2x 2 min
13	Counterstain, dehydrate and coverslip	Auxiliary	-

* See applicable IFU

7: Control slides

A positive control, negative control and reagent control are needed and processed in the same way as the unknown specimen slide to interpret staining results.

8: Storage

Store at 2-8 °C and in the dark. Do not use after expiration date.

9: Warnings and precautions

Refer to SDS.

10: Troubleshooting

Please contact WellMed by phone or by email.

11: Reference

- Shan-Rong Shi, James Guo, Richard J. cote, Lillian Young, Debra Hawes, Yan Shi, Sandra Thu and Clive R. Taylor, Applied Immunohistochemistry & Molecular Morphology, vol 7,201-208,1999